



FlexVision Reading Glasses Eyechart

IMPORTANT:

This eyechart **must be printed** in order to be accurately used. Do not attempt to use on screen as each computer displays text in a different size which will invalidate the results of the eyechart.

To determine the diopter strength that you need:

1. Print the chart.
2. Confirm the the chart is 13cm wide.
3. Hold the chart at your normal comfortable reading distance (where you would hold a book, menu or newspaper - about 30-40 cm away from your face).
4. Remove any existing reading glasses you may be wearing.
5. Starting at the top of the eyehcart, read each line and select the diopter strength next to the smallest line that you can read clearly. This is the suggested diopter strength that should be correct for you. If a line is blurry or difficult to read, move back up a line and select the next strongest diopter that you can clearly read.

ATTENTION: Ready reading non-prescription glasses may not be suitable for everyone. They are not intended to replace prescribed corrective lenses or examinations by an eyecare professional. Regular eye check-ups by a qualified professional are necessary to determine your eye health status and vision needs. Please immediately visit an eyecare professional if you have any concerns or have noticed a rapid or significant change in your vision. You should also have an annual vision checkup if you are over 40 to check for glaucoma.

Strength

- +3.00** the first line you find difficult to read
- +2.50** the first line you find difficult to read
- +2.00** the first line you find difficult to read
- +1.50** the first line you find difficult to read
- +1.00** the first line you find difficult to read

13 cm

NANNINI
— I T A L Y —

FlexVision

T: 1(300) FLEXVISION or (02) 9690-2111 F: (02) 9690-2113
E: sales@flexvison.com.au www.flexvision.com.au